

ROTANODE™
E7876X CE 0197

Rotating Anode X-ray Tube Assembly

- ◆ Rotating anode X-ray tube assembly for high energy radiographic and cine-fluoroscopic operations.
- ◆ The heavy anode is constructed with specially processed rhenium-tungsten faced molybdenum target which is 74 mm diameter and has an improved coating to increase thermal emissivity.
- ◆ This tube has foci 1.2 and 0.6, and is available for a maximum tube voltage 150 kV.
- ◆ Accommodated with IEC60526 type high-voltage cable receptacles.



General Data

IEC Classification (IEC60601-1:2005) Class I ME EQUIP MET

Electrical:

Circuit:

High Voltage Generator Constant Potential High-Voltage Generator
Grounding Center-grounded

Nominal X-ray Tube Voltage (IEC60613:2010):

Radiographic 150 kV
Fluoroscopic 125 kV

Nominal Focal Spot Value (IEC60336:2005):

Large Focus 1.2
Small Focus 0.6

Nominal Anode Input Power (at 0.1s):

	50 Hz	60 Hz
Large Focus	50 kW	54 kW
Small Focus	20 kW	22 kW

Nominal Radiographic Anode Input Power (IEC60613:2010):

	50 Hz	60 Hz
Large Focus	43 kW	48 kW
Small Focus	18 kW	21 kW

★The information contained herein is presented only as a guide for the applications of our products.
No Responsibility is assumed by TOSHIBA ELECTRON TUBES & DEVICES CO.,LTD.(TETD) for any infringements of patents or other rights of the third parties which may result from its use. No license is granted by implication or otherwise under any patent or patent rights of TETD or others.
★The information contained herein may be changed without prior notice. It is therefore advisable to contact TETD before proceeding with the design of equipment incorporating this product.

Motor Ratings:

Stator: XS-RA

		Starting	Running
Driven Frequency	[Hz]	50/60	50/60
Input Power	[W]	1450	80
Voltage ^{1) 3)}	[V]	240	58
Current ²⁾	[A]	6.5	1.5
Min. Speed Up ⁴⁾	[s]	0.6	-
Capacitor	[μ F]	24	24

Note 1) Applied voltage between common and main terminal.

2) Common current.

3) The every applied voltage must be never exceeded 110% of the above specification.

4) The speed-up time is allowed up to 110% of the above specification.

Anode Speed:

50 Hz	Minimum 2700 min ⁻¹
60 Hz	Minimum 3200 min ⁻¹

Stator Resistance:Common-Main Winding 27.5 Ω Common-Auxiliary Winding 58.0 Ω Resistance between Housing and Low Voltage Terminals Minimum 2 M Ω

Normal Operating Range of the Housing Temperature 16 ~ 75 °C

Mode of Operation Intermittent

Mechanical:

Dimensions See dimensional outline

Overall Length 479 mm

Maximum Diameter 152.4 mm

Target:

Anode Angle 12 degrees

Diameter 74 mm

Construction Rhenium-Tungsten faced Molybdenum

Filtration:

Permanent Filtration 1.3 mm Al / 75 kV IEC60522:1999

Available Additional Filter combination (0.4 - 1.5 mm) Maximum 2.8 mm Al / 75 kV

Radiation Protection (In accordance with IEC60601-1-3:2008):

Leakage Technique Factor 150 kV, 3.4 mA

X-ray Coverage 430 × 430 mm at SID 1000 mm

Weight (Approx.) 16 kg

High Voltage Receptacle To meet requirements of IEC60526 Corrigendum1:2010

Cooling Method Natural or forced air

Housing (Including both end plates):

Model Number XH-121

Absolute Maximum and Minimum Ratings (At any time, these values must not be exceeded.)

Maximum X-ray Tube Voltage (IEC60613:2010):	
Radiographic	150 kV
Fluoroscopic	125 kV
Between Anode (or Cathode) and Ground	75 kV
Minimum X-ray Tube Voltage	40 kV
Maximum X-ray Tube Current (IEC60613:2010)	See rating charts
Large Focus	700 mA
Small Focus	300 mA
Maximum Filament Current:	
Large Focus	5.4 A
Small Focus	5.2 A
Filament Voltage:	
Large Focus (At maximum filament current 5.4 A)	11.9 ~ 16.1 V
Small Focus (At maximum filament current 5.2 A)	6.8 ~ 9.2 V
Filament Frequency Limits	0 ~ 25 kHz
Continuous Anode Input Power (IEC60613:2010)	142 W (200 HU/s)
(Repeated radiographic exposure)	
Thermal Characteristics:	
Anode Heat Content	163 kJ (230 kHU)
Maximum Anode Heat Dissipation	750 W (1056 HU/s)
X-ray Tube Assembly Heat Content	¹⁾ 900 kJ (1250 kHU)
	²⁾ 975 kJ (1354 kHU)
Nominal Continuous Input Power (IEC60613:2010):	
Without Air-circulator	¹⁾ 180 W (250 HU/min)
	²⁾ 196 W (272 HU/sec)

Note 1) Based on TETD's nominal temperature range (25°C)

Note 2) Based on IEC specified maximum temperature range (20°C)

Environmental Limits



Operating Limits:	
Temperature	10 ~ 40 °C
Humidity	30 ~ 85 %
	(No condensation)
Atmospheric Pressure	70 ~ 106 kPa
Shipping and Storage Limits:	
Temperature	-20 ~ 70 °C
Humidity	20 ~ 90 %
	(No condensation)
Atmospheric Pressure	50 ~ 106 kPa

Maximum Rating Charts (Absolute Maximum Rating Charts)

Conditions: Tube Voltage
Constant Potential High-Voltage Generator
Stator Power Frequency 50 Hz

Nominal Focal Spot :Value: 1.2 ■

Nominal Focal Spot :Value: 0.6 □

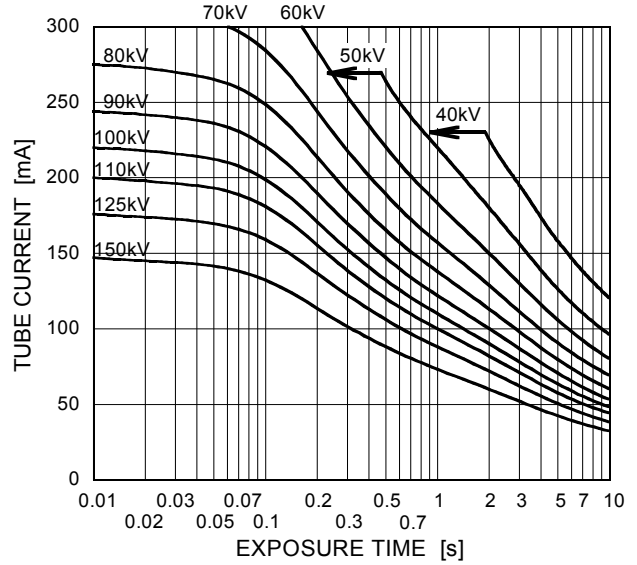
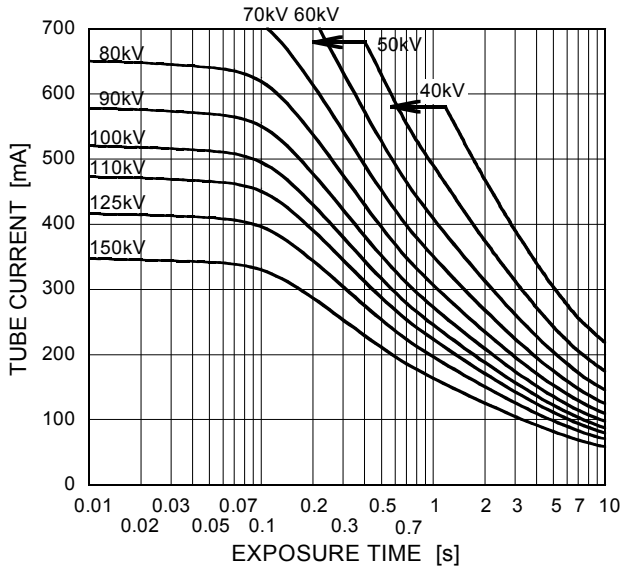


Refer to IEC60613:2010

Conditions: Tube Voltage
Constant Potential High-Voltage Generator
Stator Power Frequency 60 Hz

Nominal Focal Spot :Value: 1.2 ■

Nominal Focal Spot :Value: 0.6 □

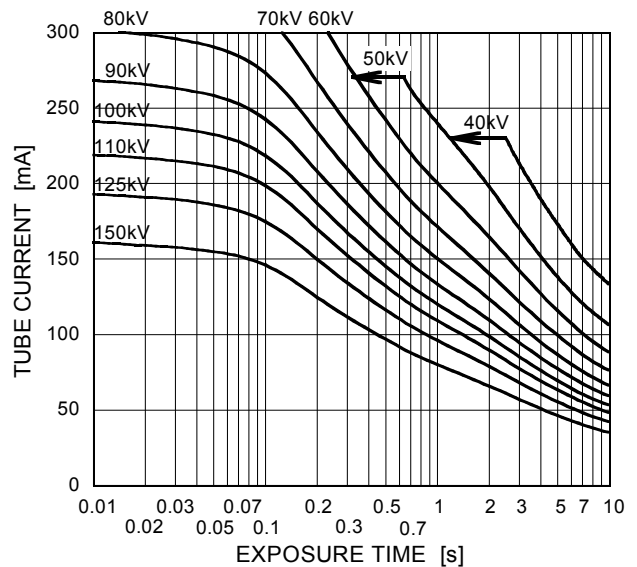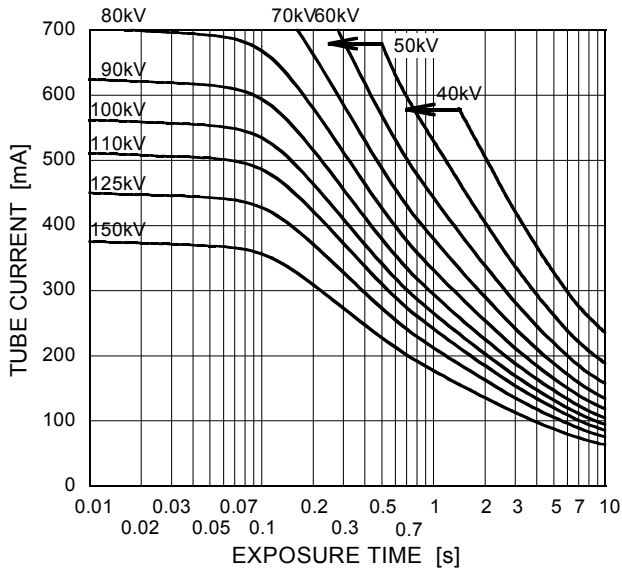


Refer to IEC60613:2010

Emission & Filament Characteristics


Constant Potential High-Voltage Generator

Nominal Focal Spot Value: 1.2 ■

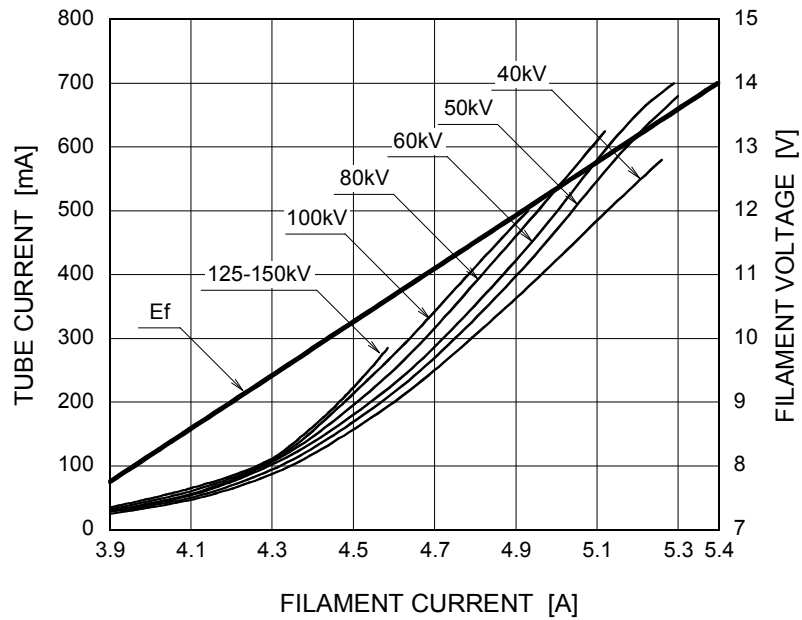

Note1) For Reference Only
 Note2) Refer to IEC60613:2010


Nominal Focal Spot Value: 0.6 □

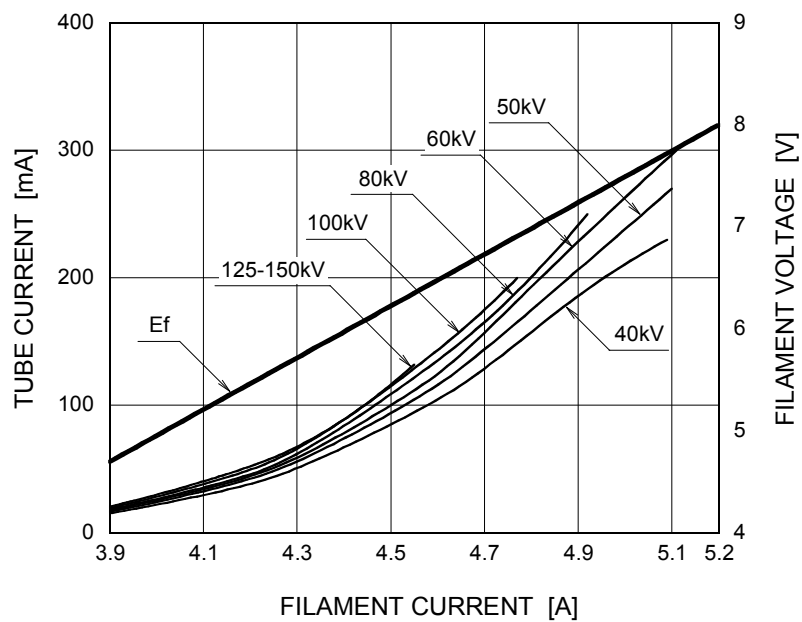


Note1) For Reference Only
 Note2) Refer to IEC60613:2010

Thermal Characteristics

X-ray Tube Assembly Heating / Cooling Curve

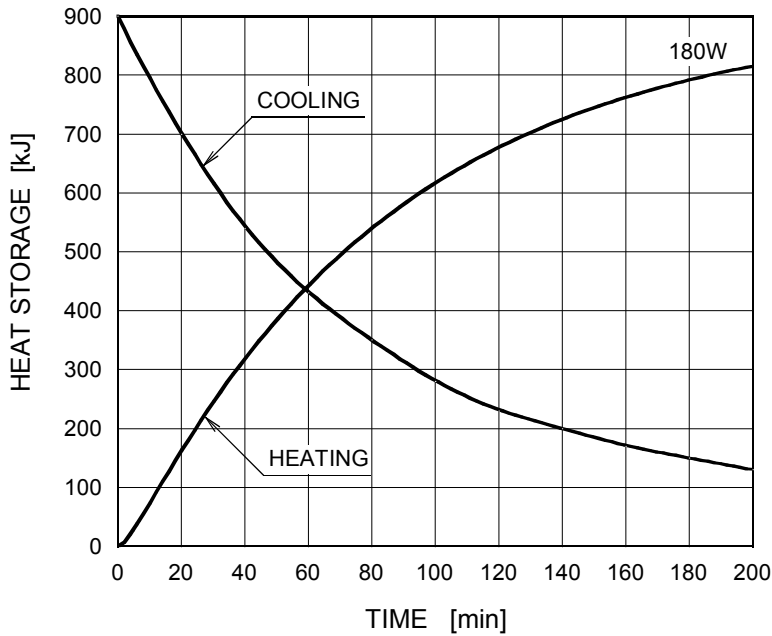


Anode Heating / Cooling Curve

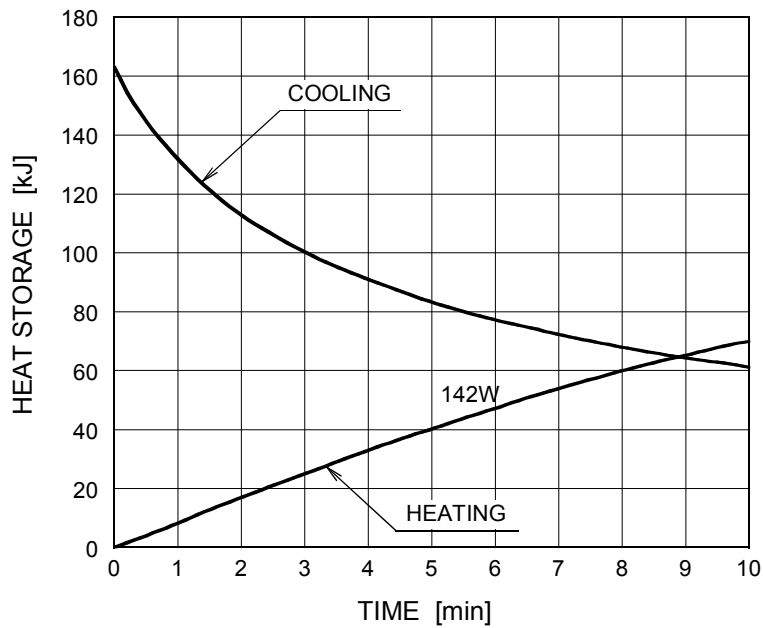

The heating curves are showing example of average input power to anode in operation.

Dimensional Outline

Unit: mm

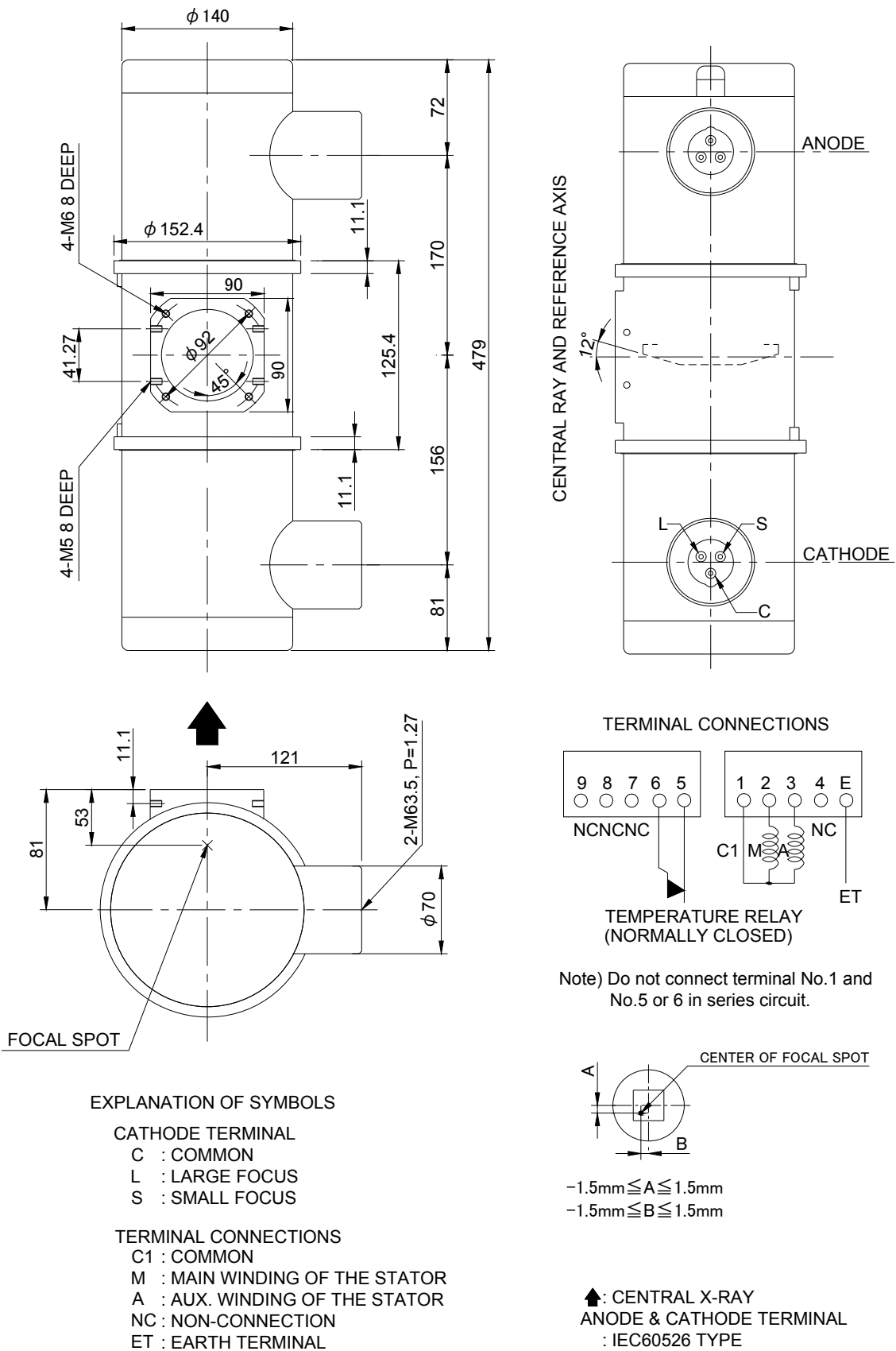

OVERSEAS SUBSIDIARIES AND AFFILIATES



EU REPRESENTATIVE

·TOSHIBA ELECTRONICS EUROPE GMBH

HANSAALLEE 181 40549 DÜSSELDORF, GERMANY

PHONE +49 (211) 5296-107 FAX +49 (211) 5296-402

For Sales & Technical Services, please contact the following representative:

·TOSHIBA ELECTRONICS EUROPE GMBH

HANSAALLEE 181 40549 DÜSSELDORF, GERMANY

PHONE +49 (211) 5296-107 FAX +49 (211) 5296-402

·TOSHIBA AMERICA ELECTRONIC COMPONENTS, INC.

2150 EAST LAKE COOK ROAD, SUITE 310

BUFFALO GROVE, ILLINOIS 60089 USA

PHONE +1 (847) 484-2400 FAX +1 (847) 541-7287

·TOSHIBA ELECTRON DEVICES & MATERIALS (SHANGHAI) CO., LTD. (TEMS)

RM1606, SH-PLAZA,

No.336, XIZANG ROAD (MIDDLE), SHANGHAI, 200001, CHINA

PHONE +86 (21) 6361-0077 FAX +86 (21) 6351-5760

 **TOSHIBA ELECTRON TUBES & DEVICES CO., LTD.**
(Product Development Engineering Department)

1385 SHIMOISHIGAMI, OTAWARA-SHI, TOCHIGI-KEN, 324-8550, JAPAN

PHONE: +81-287-26-6666 FAX: +81-287-26-6060

<http://www.toshiba-tetd.co.jp/>

·Toshiba Electron Tubes & Devices Co., Ltd. meets the Environmental Management System Standard, ISO14001.

·Toshiba Electron Tubes & Devices Co., Ltd. meets internationally recognized Standards for Quality Management System ISO9001, ISO13485.